



SUTURE CART

FOUR SLANTED SHELF UNIT
DC15EC, DC35EC, DC55EC

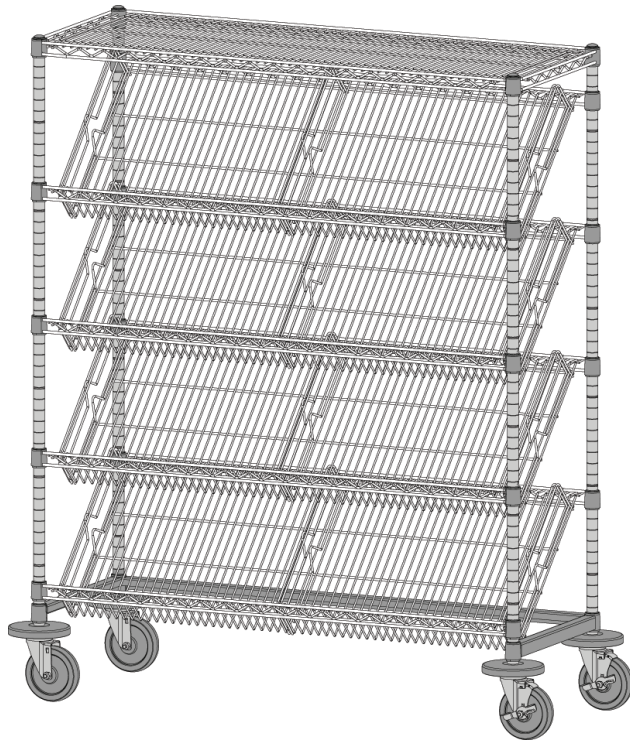

FIVE SLANTED SHELF UNIT
DC16EC, DC36EC, DC56EC

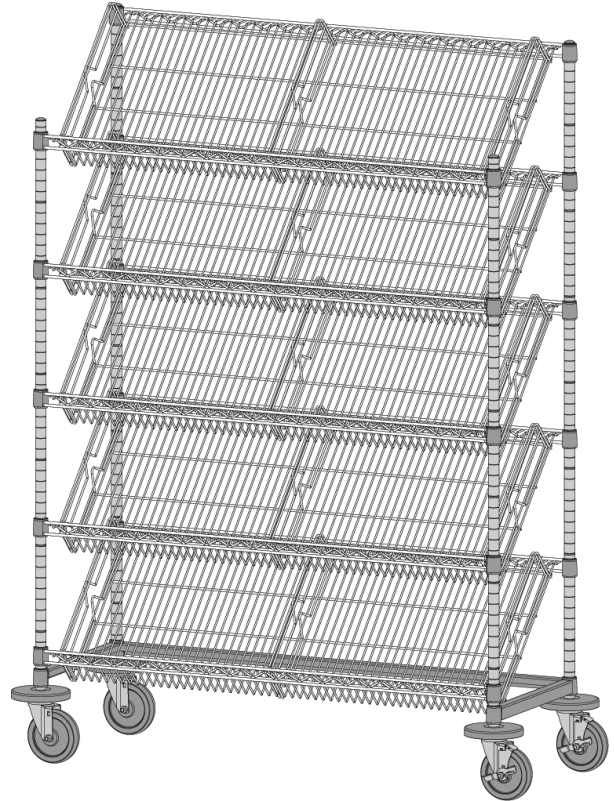


InterMetro Industries Corporation
North Washington Street, Wilkes-Barre, PA USA 18705
For Product Information Call: 1-800-992-1776
Visit Our Web Site: www.metro.com

**ASSEMBLY
INSTRUCTIONS**
For Suture Cart


Suture Cart

ASSEMBLY
INSTRUCTIONS FOR MODELS
DC15EC, DC35EC, DC55EC,
DC16EC, DC36EC & DC56EC
Two People Required to Assemble Unit.

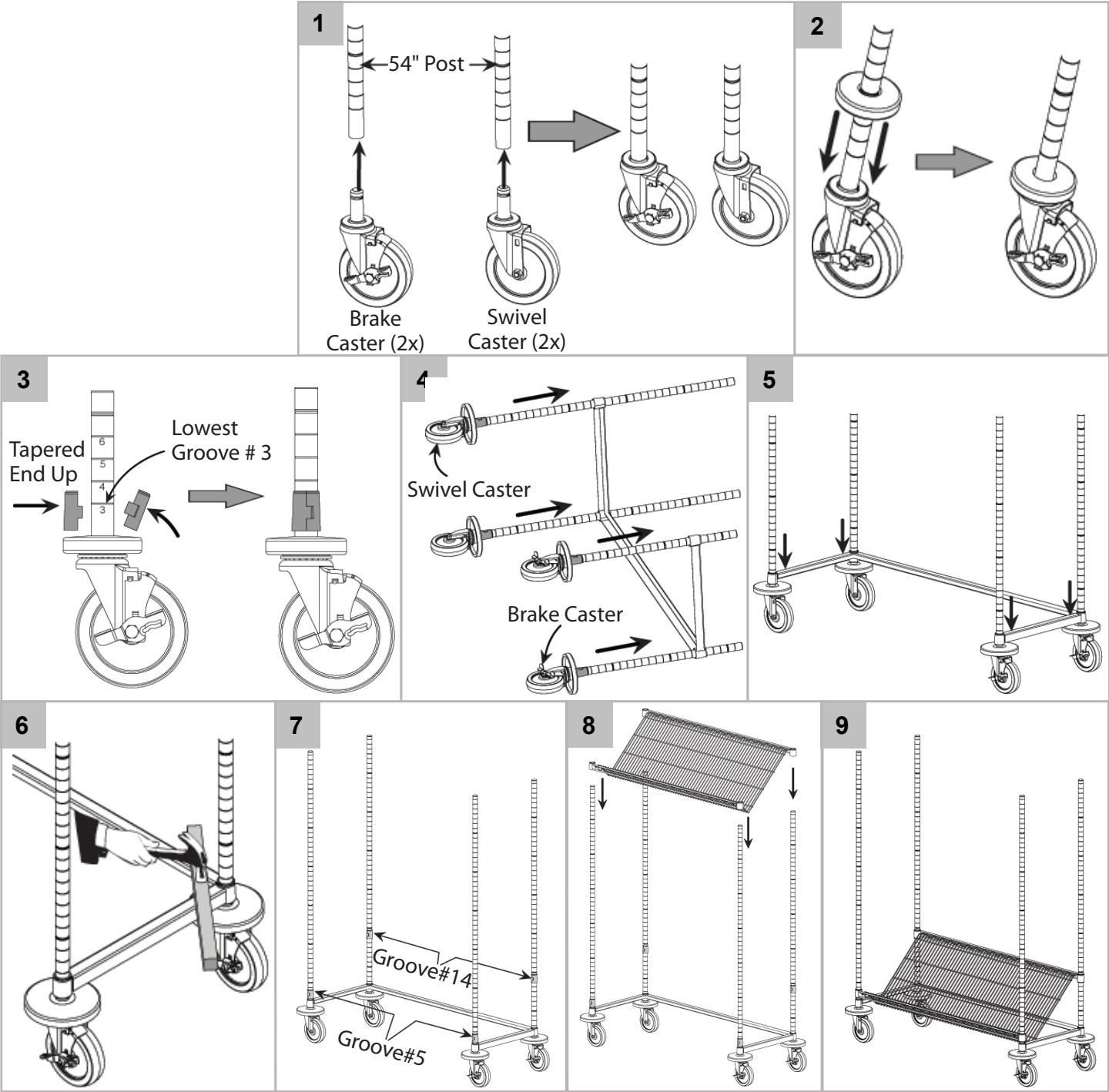


10 REPEAT STEPS 5-6


11

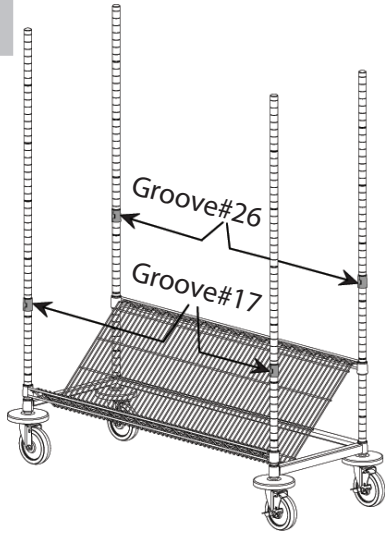

12

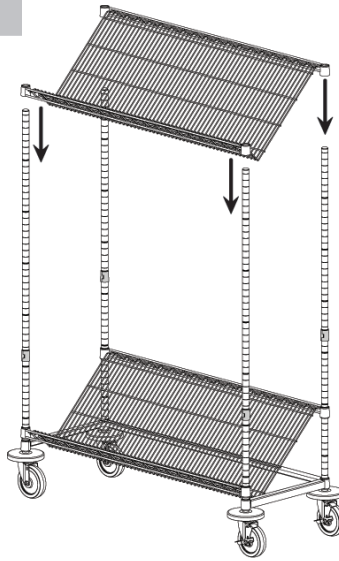


13

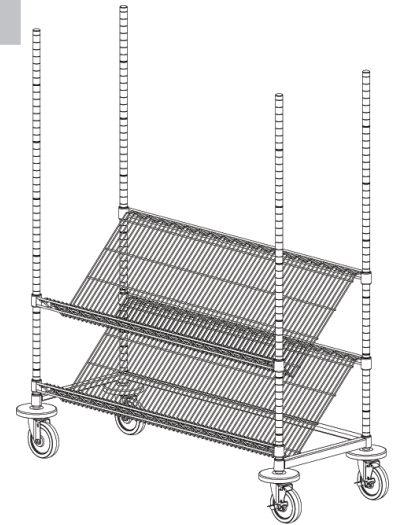


14 REPEAT STEPS 5-6


15

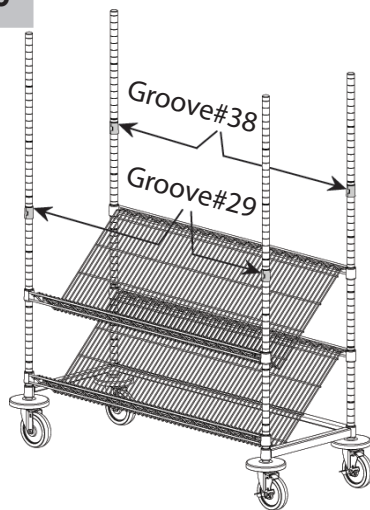

16

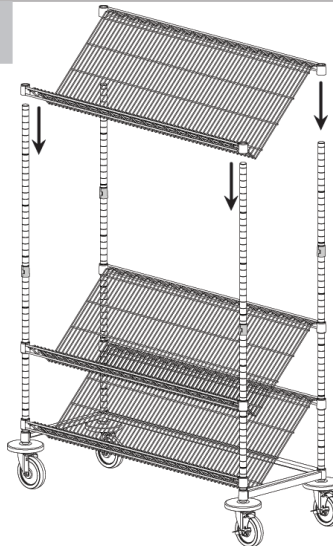


17

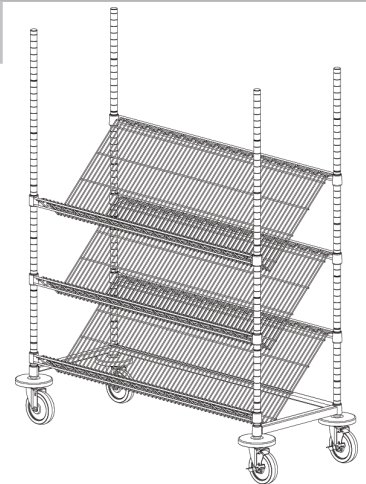


18 REPEAT STEPS 5-6


19

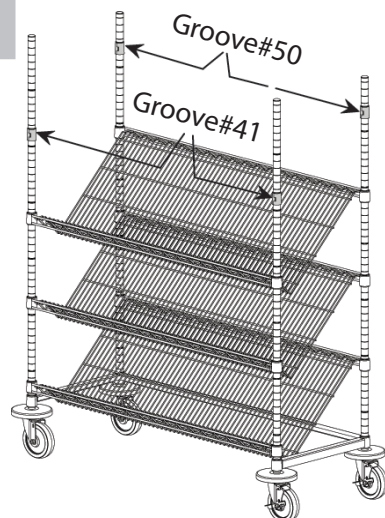

20

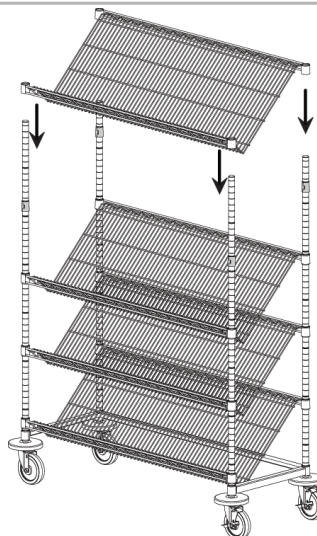

21

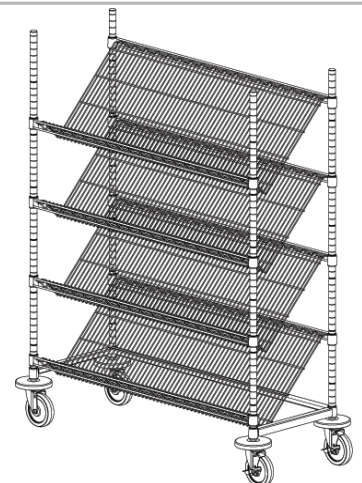


22 REPEAT STEPS 5-6


Suture Cart

23

Four Slanted Shelf

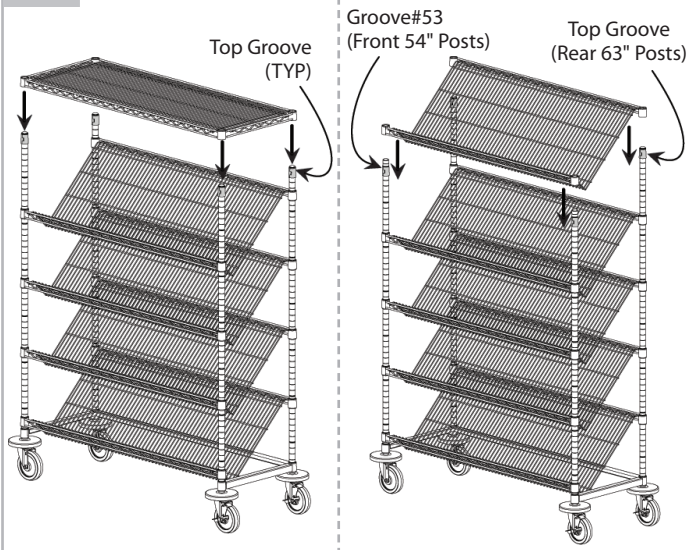


24

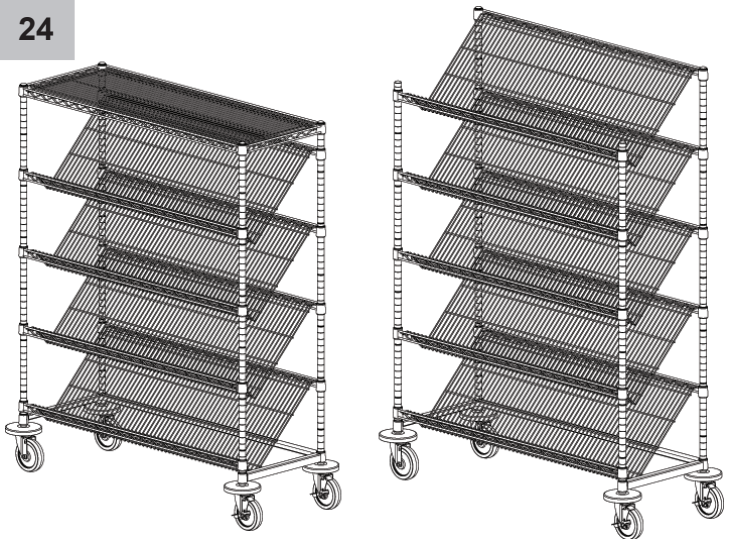

25

REPEAT STEPS 5-6

26

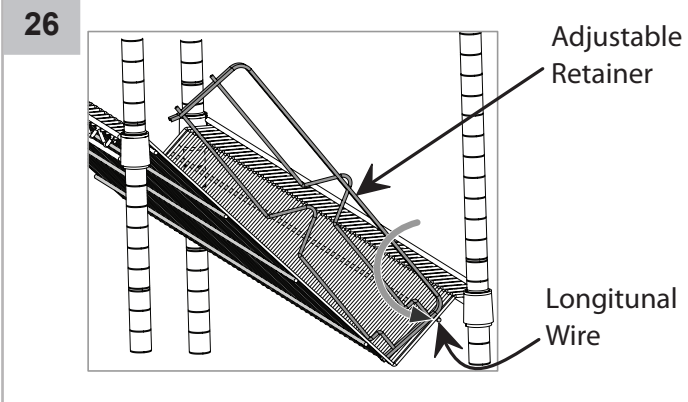


Adjustable Retainer


Longitudinal Wire

27

Squeeze the Retainer at the Center

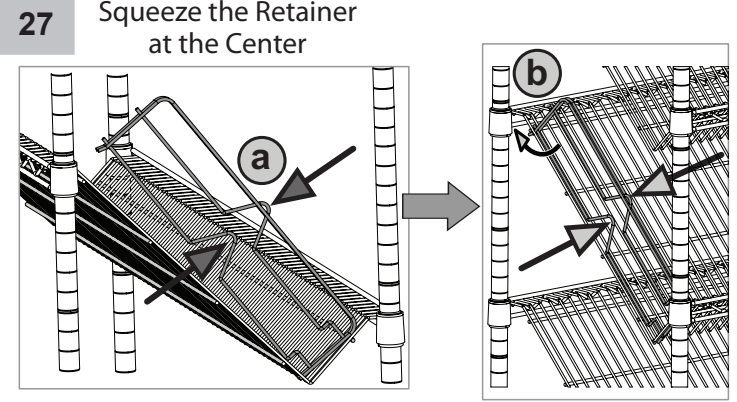

28

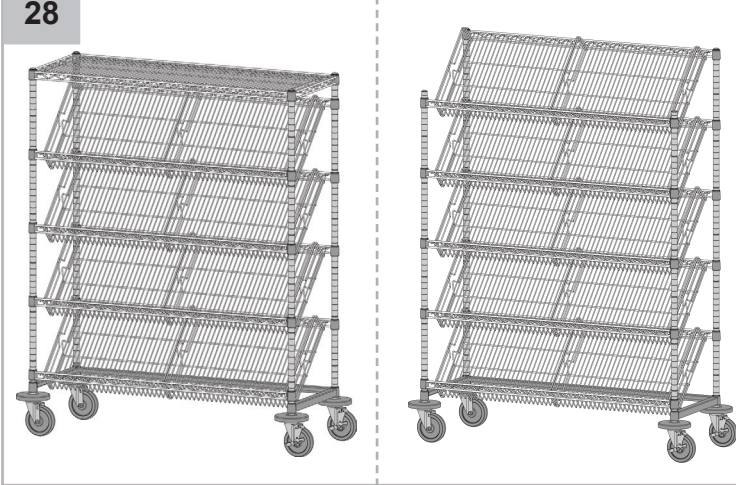

THIS COMPLETES THE ASSEMBLY OF FOUR AND FIVE SLANTED SHELF UNITS



25 lbs.
Max Load
Per Shelf

InterMetro Industries Corporation
North Washington Street, Wilkes-Barre, PA USA 18705
For Product Information Call: 1-800-992-1776
Visit Our Web Site: www.metro.com

L01-069
Rev. A
05/11
Information and specifications are subject to change without notice. Please confirm at time of order.