

## **PLEASE READ WARNINGS AND DIRECTIONS BEFORE USING:**

Formulated Medical Cyanoacrylate Manufactured under U.S. Patents 6155265 and 6386203.

### **DIRECTIONS:**

**Preparation:** Wash area thoroughly and make sure it is clean and dry. Do not apply any ointments, creams or lotions before or after application of the bandage.

**Application:** 1) Remove overcap, and gently twist off tip. 2) Topically apply an appropriate amount of Liquid Skin to the desired area. 3) Setting time will vary by thickness of application from 30 seconds to several minutes. If placed in undesirable area, it can be easily wipe off with a tissue. 4) Discard pipette after use.

**Removal:** Product will slough off naturally as the dermal layer continues to grow. If necessary, it can be removed by applying lotions, ointments or emollients and peeled off.

**Storage:** Store at room temperature (Do Not refrigerate).

**WARNINGS:** Do not use in or around the eyes or on burns, wounds or skin with active signs of infection. Avoid placing adhesive into lacerations. Consult your doctor for use in case of a serious wound, burn or animal bite. Make sure the bandage has fully dried before allowing it to come in contact with other objects.

**KEEP OUT OF REACH OF CHILDREN. CAUTION:** Should not use if allergic to cyanoacrylate. Use as directed.

Manufactured by Chemence, Inc.  
185 Bluegrass Valley Parkway, Alpharetta, GA 30005

Distributed by Medique Products  
17080 Alico Commerce Ct, Fort Myers, FL 33967